

Dry socket?
Ease the pain.



Alveogyl

Dry socket dressing



Alveogyl is the ideal dry socket dressing, rapidly easing your patient's pain and convenient for you to use. Alveogyl may also be used as a post-extraction dressing.

Alveogyl is a one-step, self-eliminating treatment requiring no suturing and no special attention other than observation of the healing process. It provides a soothing effect on the alveolar tissues thus helping to rapidly alleviate the pain. Its fibrous consistency, due to the Penghawar fibers, allows an easy filling of the socket and a good adherence to the alveola.

| Features | Advantages | Benefits |
|----------------------------|--|---|
| Contains Penghawar fibers | Fibrous consistency | Good adherence to the alveolus during the entire healing process |
| Dry socket dressing | Promotes hemostasis by compression | Helps to rapidly alleviate the pain Protects from superinfection |
| No suturing required | No special attention other than observation of the healing process | Convenient and easy to use |
| Self eliminating treatment | One-step procedure | No need for recall appointments |

Indications

Alveogyl is a paste used as dressing in case of dry socket or post-extraction dressing following a difficult or traumatic extraction in patients with history of dry socket.

Instructions for use

- Take a small pellet (about 0.20 g) of Aveogyl paste
- Place it gently into the prepared dental socket
- Do not suture

Product Information

10 g Jar

