

INSTRUCTION FOR USE

DEVITALIZE PLUS 2.5G

Pulp Devitalization paste

COMPOSITION

Racemic Camphor, Polyoxymethylene, ParaChlorophenol, Lidocaine Hydrochloride, Paste-Forming Component, Colorant, Fiber filler

PROPERTIES

Arsenic-free Devitalization fibrous paste.

Polyoxymethylene is an antiseptic, which coagulates proteins providing necrosis of pulp tissue. Lidocaine Hydrochloride is an anesthetic component, reducing pain risk during tooth Pulp Devitalization. Racemic Camphor along with ParaChlorophenol make next generation pulp devitalization paste; which gives synergetic effect to Lidocaine Hydrochloride and Polyoxymethylene. DEVITALIZE PLUS 2.5 provides prolonged and painless pulp devitalization within 1-5 days.

Paste is intended for use in aggressive biological medium with the temperature range between 32 °C and 42 °C (in oral cavity at the temperature (37±1) °C and humidity 60-90 %).

INDICATIONS

Pulp Devitalization before its extirpation

CONTRAINDICATIONS

Drug idiosyncrasy. Do not misuse.

Use with great care for patients allergic to Lidocaine hydrochloride.

Do not swallow.

Warning! In case of paste leakage through the temporary filling (scent and flavor of clove oil appears) the patient should immediately consult a specialist.

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects.

METHODS OF USE

Ease to handle application of paste with Syringe Delivery System.

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com




info@safeendo.org

Close cap after every use.

Remove Carious Infected Dentine with the help of Dental Excavator. Using special instrument apply necessary amount of paste (2 mm ball) on tooth pulp horn or close to it. Fill with temporary dressing without compressing using paste SmartTemp (SafeEndo). After pulp Devitalization, if no pain left, pulp extirpation with further root canal treatment procedures can be fulfilled.

Warning! In case of increased pain after applying of the paste (tooth pulp horn is not open or paste is applied with too much compression) make an infiltration anesthesia with Lidocaine Hydrochloride and apply paste closer to the pulp.

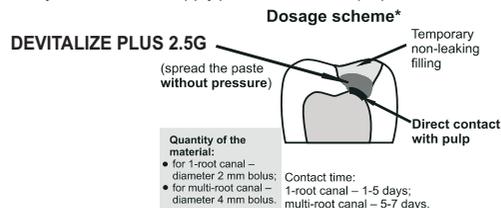

PRESENTATION

Paste (plastic jar)

8 g Ref no 004-02-250

STORAGE

Store at the temperature from 5 °C to 25 °C.

Tightly close container immediately after use.

Do not use after expiry date.

Shelf life – 3 years.

MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: info@safeendo.org • www.safeendo.org

Not For Medicinal use, For use in Dentistry By Dental Professional only

004-02-250

Date of last revision: 2021-02

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org