

INSTRUCTION FOR USE

SocketCure

Paste with Iodoform

COMPOSITION

Butamben, Iodoform, Eugenol, excipients

PROPERTIES

Nonresorbable temporary Socket dressing paste.

SocketCure is a fiber paste with specific odor used for antiseptic treatment of extraction socket.

SocketCure contains :

- Butamben is a local anesthetic, acts by inhibiting the voltage-gated calcium channels in dorsal root ganglion neurons.
- Iodoform – antiseptic, possesses durable antibacterial effect
- Eugenol, enhances antiseptic effect and thanks to its analgesic benefit, relieves pain
- excipients, provide sustained therapeutic effect

SocketCure is not resorbable. Paste washes out of socket over time, which helps in wound healing. During follow-up there's no need to extract paste remains, i.e. no soft tissue traumatizing.

Dental product is produced for using in aggressive biological medium with the temperature range between 32°C and 42°C (in oral cavity at the temperature $(37\pm 1)^\circ\text{C}$ and humidity 60-90 %).

INDICATIONS

SocketCure is used during:

- Socket treatment after tooth extraction
- Alveolgia prophylaxis
- Acute alveolitis treatment

CONTRAINDICATIONS

Iodine or its derivatives allergy

Do not use for patients with blood-clotting disorder

Do not swallow

SIDE EFFECTS

Can be the reason for skin irritation when in contact. Use gloves during work.

As far as all terms of storing, transporting and application are observed, there are no


other side effects

METHOD OF USE

Lay on a bandage roll small amount of paste and place it in socket for 2-6 days, or, make a small pellet of paste and place it in socket.

Do not suture, paste is not resorbable.

The medication works for several days. That is important to give patient a warning not to swallow the paste and not to rinse mouth hard.

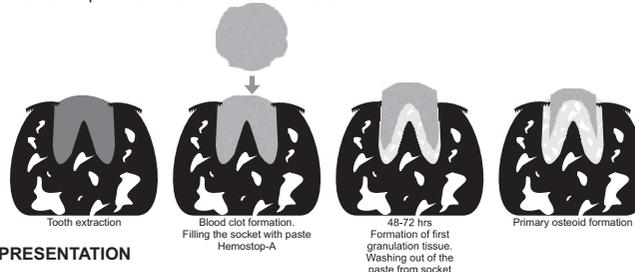

PRESENTATION

Paste (jar) 15g 004-05-015

STORAGE

Store at the temperature from 5°C up to 25°C

Tightly close the jar after use

Do not use after expiry date

Shelf life – 3 years

MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate,

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: info@safeendo.org

www.safeendo.org

Not For Medicinal use, For use in Dentistry By Dental Professional only

004-05-015

Date of last revision: 2021-06

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org