

## INSTRUCTION FOR USE

### DisSolve E

#### COMPOSITION

D-Limonene based Gutta Percha and Zinc Oxide Eugenol Cement Solvent

#### PROPERTIES

DisSolve E, Gutta Percha Solvent is used for removing gutta percha at the time of root canal re-treatment. Also useful for removing all types of root canal filling materials including Zinc Oxide – Eugenol Cement or as a cleaner for endodontic instruments. DisSolve E, Gutta Percha Solvent is based on natural substances and therefore it does not cause irritation to oral mucosa and peri-apical tissue.

Dental product is produced for using in aggressive biological medium with the temperature range between 32°C and 42°C (in oral cavity at the temperature (37±1)°C and humidity 60-90 %).

#### INDICATIONS

DisSolve E G.P. Solvent is used for removing gutta percha at the time of root canal retreatment. Also useful for removing all types of root canal filling materials including Zinc Oxide – Eugenol Cement or as a cleaner for endodontic instruments.

#### CONTRAINDICATIONS

Drug idiosyncrasy

Use with great care for allergic patient

Don't misuse

DisSolve E Gutta Percha Solvent is volatile and flammable. Keep away from heat source. Keep bottle tightly capped after every use.

#### SIDE EFFECTS

No known side effects as far as all terms of application, transportation and storage are observed. Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation.

#### METHOD OF USE

Dispense DisSolve E Gutta Percha Solvent into dappen dish or directly on cotton pallet. Place the soaked cotton pellet on root canal filling material and cover with temporary cement (Zinc Phosphate). Remove the softened filling material after two days.

#### PRESENTATION

DisSolve E liquid 15ml Ref no 002-06-015

#### STORAGE

Store at 5 °C to 25 °C, in a well sealed container

Attention! Close materials tightly after use

The material must not be used after the expiry date

Shelf life – 3 years

#### MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: [info@safeendo.org](mailto:info@safeendo.org)

[www.safeendo.org](http://www.safeendo.org)

*Not For Medicinal use, For use in Dentistry By Dental Professional only*

002-06-015

Date of last revision: 2021-01