

## INSTRUCTION FOR USE

### CresoPulp

#### COMPOSITION

Chlorophenol, Lidocaine hydrochloride, Eugenol, M-Cresol, Camphor,

#### PROPERTIES

Lidocaine hydrochloride provides analgesic effect. Chlorophenol, eugenol and M-cresol are broad spectrum antiseptics, possessing slightly mummifying effect on nerve fibers. Camphor mitigates the impact of chlorophenol on the body.

CresoPulp Liquid is intended for use in aggressive biological medium with the temperature range between 32°C and 42°C (in oral cavity at the temperature (37±1)°C and humidity 60-90 %).

#### INDICATIONS

CresoPulp is used:

- For anesthesia in traumatic and acute pulpitis before applying of devitalization paste (Devitalize)
- As a long-acting application for pulpitis management

#### CONTRAINDICATIONS

Drug idiosyncrasy  
Do not misuse

#### SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects

#### METHODS OF USE

CresoPulp should be applied with cotton swab or cotton ball for applications, saturated with liquid and slightly squeezed in order to avoid leakage of liquid.

During treatment of deep caries and acute pulpitis insert saturated swab in the cavity, isolate with temporary material (SmartTemp) without compressing and leave for 1-3 days.

During long-term treatment, using CresoPulp as a long-acting unpenetrable application, put saturated cotton ball under a temporary filling (SmartTemp). In a period (up to 10 days), determined by dentist, planned treatment procedures can be fulfilled.

#### PRESENTATION

CresoPulp (bottle) 15ml 004-03-015

#### STORAGE

Store at the temperature from 5°C to 25°C. Keep in dry place

Tightly close container immediately after use

Do not use after expiry date

Shelf life – 3 years

#### MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate,

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: [info@safeendo.org](mailto:info@safeendo.org)

[www.safeendo.org](http://www.safeendo.org)

***Not For Medicinal use, For use in Dentistry By Dental Professional only***

004-03-015

Date of last revision : 2021-06

**SafeEndo**  
enhance your endo



+91 79-22139977 / 22163335



[www.safeendo.org](http://www.safeendo.org)  
[www.safeendoindia.com](http://www.safeendoindia.com)



[info@safeendo.org](mailto:info@safeendo.org)

**SafeEndo**  
enhance your endo



+91 79-22139977 / 22163335



[www.safeendo.org](http://www.safeendo.org)  
[www.safeendoindia.com](http://www.safeendoindia.com)



[info@safeendo.org](mailto:info@safeendo.org)