

INSTRUCTION FOR USE

ReTemp - Lc 2.5g Blue

Light Curing Temporary Filling flowable Composite

COMPOSITION

Acrylated Urethane Photoinitiators BHT, Thickeners

ReTemp – Lc is a blue colour, light curing paste based on a polymer binder and inorganic filler. Thanks to its elastic consistency, the material is easily removable, also in case of undercuts with help of appropriate hand instruments (probe or excavator) with a single shape of cavity. Therefore post-treatment drilling of cavity to remove temporary material is not needed. ReTemp - Lc Flow is a light curing flowable composite designed for temporary fillings in endodontic treatment, micro prosthesis, temporary sealing of abutment screw of Implant

Properties and Method of Application

ReTemp – Lc is inserted with applying needle cannula (recommended single use) directly into prepared cavity area, Photo polymerization of the material is conducted with ultra violet light of 400-500 nanometer wave length for 20-40 seconds depending upon thickness of layer from 1mm-3mm. Check occlusion and remove the excess material with help of silicone disc or appropriate finishing tools.
No Need of Etching and Bonding Required.

INDICATION

- Temporary filling of Inlay and Onlay of the cavity
- Sealing of Implant Abutment Screws
- Block out material for retentive areas in the dental arch
- Covering of the Gingival margin
- Temporary preparation of Cavity wall for Endodontic irrigation

SafeEndo
enhance your endo

+91 79-22139977 / 22163335  www.safeendo.org  info@safeendo.org
www.safeendoindia.com

- Sealing of Root Canal Orifices after obturation

CONTRAINDICATION

Drug idiosyncrasy

Use with care if person is allergic to methacrylates and BHT

Do not use directly on exposed Pulp area as a temporary or permanent closure

SIDE EFFECTS

No any known side effects as far as all terms of application, transportation and storage are observed.

PRESENTATION

ReTemp – Lc 2*2.5g syringe Ref No. 005-02-250

STORAGE

Store at 5 °C to 25 °C, in a well sealed container Attention! Close materials tightly after use

The material must not be used after the expiry date Shelf life – 2.5 years

MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate,

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: info@safeendo.org

www.safeendo.org

Not For Medicinal use, For use in Dentistry By Dental Professional only

005-02-250

Date of last revision : 2021-04

SafeEndo
enhance your endo

+91 79-22139977 / 22163335  www.safeendo.org  info@safeendo.org
www.safeendoindia.com