

## INSTRUCTION FOR USE

### SteriPulp

#### COMPOSITION

Formaldehyde, Phenol, filler

#### PROPERTIES

Phenol and Formaldehyde are antiseptics denaturalizing microorganisms' proteins, thus providing antiseptic effect, deep mummification and root canals sterilization.

Liquid is produced for using in aggressive biological medium with the temperature range between 32°C and 42°C (in oral cavity at the temperature (37±1)°C and humidity 60-90 %).

#### INDICATIONS

SteriPulp is used as antiseptic for treatment of infected root canals in complicated pulpitis and resection of upper part of tooth root

#### CONTRAINDICATIONS

Drug idiosyncrasy

Do not misuse

#### SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects

#### METHOD OF USE

SteriPulp should be applied with the use of saturated and carefully

squeezed cotton swabs, avoiding any leakage of the liquid, and left in tooth canal orifice for 2-3 days.

Saturated with SteriPulp cotton swabs can be used for treatment of infected root canals or as a temporary application.

In 2-3 days pull out a cotton swab, treat the canal one more time and fill the sterile root canal with sealer EugeSeal

#### PRESENTATION

SteriPulp (bottle) 15ml 004-04-015

#### STORAGE

Store at the temperature from 5°C to 25°C

Keep in dry place

Tightly close container immediately after use

Do not use after expiry date

Shelf life – 3 years

#### MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate,

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: [info@safeendo.org](mailto:info@safeendo.org)

[www.safeendo.org](http://www.safeendo.org)

*Not For Medicinal use, For use in Dentistry By Dental Professional only*

004-04-15

Date of last revision : 2021-06

**SafeEndo**  
enhance your endo



+91 79-22139977 / 22163335



[www.safeendo.org](http://www.safeendo.org)  
[www.safeendoindia.com](http://www.safeendoindia.com)



[info@safeendo.org](mailto:info@safeendo.org)

**SafeEndo**  
enhance your endo



+91 79-22139977 / 22163335



[www.safeendo.org](http://www.safeendo.org)  
[www.safeendoindia.com](http://www.safeendoindia.com)



[info@safeendo.org](mailto:info@safeendo.org)