Cleaning Performance Test

Chemdye® Splat CDWAH Holder

(A) BASKET ZONE: Water pressure, spray system performance, location and other mechanical functions.

(B) COVERED ZONE: Detergent chemistry and concentration, water quality, temperature, cycle duration, etc.

| Results reference guide in Washer-Disinfectors (OK / X FAIL) | | | | | | | |
|--|-------------------------------|----------------|--|--------------------------------|---------------|------------------------|---------------|
| Visual results | BASKET ZONE influence results | | | COVERED ZONE influence results | | | |
| | Location inside washer | Water pressure | Mechanical function (detergent pump/spray arms etc.) | Temperature | Exposure time | Detergent (type, dose) | Water quality |
| Processed and correct | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | ✓ | ✓ | ✓ | ✗ | ✗ | ✗ | ✗ |
| | ✓ | ✓ | ✓ | XX | XX | XX | XX |
| | ✓ | ✓ | ✓ | XXX | XXX | XXX | XXX |
| | ✗ | ✗ | ✗ | XXX | XXX | XXX | XXX |
| | ✗ | ✗ | ✗ | XXX | XXX | XXX | XXX |
| | XXX | XXX | XXX | XXX | XXX | XXX | XXX |
| Unprocessed | — | — | — | — | — | — | — |

NOTE: This reference table corresponds to results when using Chemdye® Splat CDWA4 in Washer-Disinfectors and could be a consequence of failure in one or a combination of variables. When detecting the failed variable/s, make the corrective actions. If the problem persists, contact the technical service of your washing machine. The higher the quantity of **X**, the higher the influence of that variable on the failure of the cycle. Same results could be obtained when using Chemdye® Splat CDWA3 under the user's specific conditions.

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Results reference guide for Ultrasonic Washers

| Grade | % Ink residues | Visual results |
|-------|----------------|-----------------------|
| 0 | 0 % | Processed and correct |
| 1 | 0 - 20 % | |
| 2 | 20 - 40 % | |
| 3 | 40 - 60 % | |
| 4 | 60 - 80 % | |
| 5 | 80 - 100 % | |

Explanation of Symbols

LOT Batch number.

MF Manufacture Date.

EX Expiration Date.

MAN Manufacturer.

ATTENTION: If your chemical indicator has this logo printed on it, means that your product is compatible with exclusive Trazanto Automatic Quality Control and Traceability System for cleaning, disinfection and sterilization processes. This System interprets the indicator's results in an accurate and automatic manner, and stores all the process data through Biornova® Software. Use **Trazanto** scanner to interpret the results (see instructions for use in the manual of the device).

ATTENTION: Do not write on the indicator before scanning.

Chemdye® Splat CDWAH-U Holder

Only for ultrasonic cleaning performance test.

NOTE: Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators. Images are reproduced for illustration purposes only, and do not necessarily match the article described in every detail.

Results reference guide in Washer-Disinfectors (OK / X FAIL)

| Visual results | BASKET ZONE influence results | | | COVERED ZONE influence results | | | |
|-----------------------|-------------------------------|----------------|--|--------------------------------|---------------|------------------------|---------------|
| | Location inside washer | Water pressure | Mechanical function (detergent pump/spray arms etc.) | Temperature | Exposure time | Detergent (type, dose) | Water quality |
| Processed and correct | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ | ✓ |
| | ✓ | ✓ | ✓ | ✗ | ✗ | ✗ | ✗ |
| | ✓ | ✓ | ✓ | XX | XX | XX | XX |
| | ✓ | ✓ | ✓ | XXX | XXX | XXX | XXX |
| | ✗ | ✗ | ✗ | XXX | XXX | XXX | XXX |
| | ✗ | ✗ | ✗ | XXX | XXX | XXX | XXX |
| | XXX | XXX | XXX | XXX | XXX | XXX | XXX |
| Unprocessed | — | — | — | — | — | — | — |

NOTE: These results correspond to examples of possible results when using either Chemdye® Splat CDWA3 or Chemdye® Splat CDWA4 with the corresponding holder for ultrasonic cycles, Chemdye® Splat CDWAH-U. The examples shown are only for approximations for user guidance.

EN Cleaning Indicators
For cleaning performance tests

Control of cleaning and washing processes of medical instruments and materials is highly important since the outcome affects subsequent disinfection and/or sterilization processes success. Cleaning control is particularly relevant in automatic washing machines, where even though process parameters may be well controlled, the cleaning final result depends largely on uncontrolled factors. Washing efficacy is influenced by detergent, water temperature, time and washing penetration. A good cleaning result is achieved by an appropriate combination of these variables. The same considerations apply for washing processes in ultrasonic washing machines.

Product description

Chemdye® Splat CDWA indicators consist of a 70 x 30 mm temperature stable support containing a mixture of specially combined natural colored components (proteins, carbohydrates, fatty acids, and dyes). In all cases, indicator colored test soil is non-toxic, dissolves with most of the cleaning agent and is swept along with the washing fluid. Chemdye® Splat CDWA cleaning indicators formula allows detection of all the factors that affect the outcome of the washing process. Chemdye® Splat CDWA indicators can be used for routine control of cleaning processes and allow monitoring of washing effectiveness in automatic machines including ultrasonic washing machines with different washing configurations.

Chemdye® Splat CDWA cleaning system offers two different levels of cleaning with different adhesion characteristics, thus allowing different challenge levels to test cleaning efficacy. The indicators efficiency is validated by comparing against the test soils detailed in ISO/TS 15883-2:2005 standard annexes.

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*1: The tests carried out with Chemdye® Splat CDWA indicators in ultrasonic washing machines were performed using the holder designed for this purpose, Chemdye® Splat CDWAH-U, which allows to fasten the indicator in a specific

location inside the ultrasonic bath, without representing a challenge to the washing processes.

*2: Enzymatic detergent, tested at a standard concentration of 0.5% V/V.

*3: Alkaline detergent, tested at a standard concentration of 0.2% V/V.

*4: Products were not tested with enzymatic detergent in the range between 50-70 °C, because most enzymatic detergents may not work efficiently at this temperature range.

Note: It is recommended to test the 2 types of Chemdye® Splat CDWA indicators along with the instrument, in order to select the one which is more appropriate for the washing machine performance. It is advisable to start the washing control validation process using the most robust Chemdye® Splat CDWA product. The indicator that washes completely during the washing control validation process should be used for routine monitoring.

It should be noticed that the choice of indicator will also depend on the hardness of the object to be washed. Therefore the Chemdye® Splat CDWA indicator selected for routine monitoring of a particular washing machine could be changed according to the hardness of the soil on the load. Once the Chemdye® Splat CDWA indicator appropriate for the washing requirements has been selected, we recommend placing several identical indicators at different positions of the washing machine, so as to monitor the performance of all areas of the machine during the washing process.

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