BT10

Self-Contained Biological Indicator

For Ethylene Oxide (EO) sterilization processes.

Usage

Monitoring Ethylene Oxide Sterilization Processes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11138-1:2017 and ISO 11138-2:2017; IRAM 37102-1:1999 and IRAM 37102-2:1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

Polypropylene tube: 45.2 mm high x 8.5 mm external diameter. Wall thickness: 0.5 mm.

Polypropylene cap: 13.0 mm high x 10.9 mm external diameter. Wall thickness: 0.7 mm.

Cap filter: medical grade paper, 17.0 mm diameter, 60.0 g/m².

Glass ampoule: 35.0 - 40.0 mm high. External diameter: 6.5 mm. Wall thickness: 0.2 - 0.3 mm.

Culture medium: 0.5 - 0.7 ml, **blue** color.

Spore carrier: filter paper, 7.0 mm diameter, 160.0 g/m².

10⁶ *Bacillus atrophaeus* ATCC 9372 spores per vial.

Positive growth results (failure in the sterilization process): Culture medium turns to **yellow**, incubation at 37 °C for 48 hours (maximum).

D-Value: Not lower than 2.0 minutes, conditions: 600 ± 30 mg/l EO, 60 ± 10 % RH, 54 ± 1 °C.

Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH 30-80 %, keep in a dark place.

Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

2 years.

Packing

30 or 100 units per box.

Packing information: product code and description, process for intended use, presentation, regulation, bacterial strain, storage conditions, manufacturer information and data on box's label.

Labelling

On product: 17.0 mm x 33.0 mm polypropylene label. Printed in black. 1.5 mm chemical indicator line, printed with EO reactive ink (color change to green). Product code, process for intended use, name of organism, batch number and expiration date.

On product's packing: product code, batch number, bacterial load, manufacture and expiration date, bar code and datamatrix code.

NOTE: manufacture date is calculated by subtracting 2 years to the expiration date.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 37 °C in BIONOVA® IC10/20 incubator. Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to Steam, Dry Heat, Radiation or any sterilization process other than EO.

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